## CarePocket® Premium

### EN Leg Bag Holder

**Indication:**  
Medical device for retention of urine leg bag.

**Features:**  
CarePocket® Premium provides a pocket for retention of a urine leg bag. It has a slit at the bottom for the valve of urine bags with bottom tap opening. The sleeve can contain urine leg bags of volumes up to 750 ml/24 oz. Suitable for use on both thigh and calf. With outer seam.

**Precautions:**  
It is recommended to empty the urine bag when two-thirds full. If you have any questions regarding use and/or correct fitting, or if any unexpected discomfort or skin irritation occurs, please consult your healthcare professional.

#### Instruction for use:

1. Pull on the leg bag holder. Ensure that the two coloured lines are at the top.
2. Pull down the outer layer of the sleeve and place the leg bag in the pocket.  
Pull the tap through the hole at the bottom of the sleeve.  
Pull up the outer layer of the sleeve.
3. The CarePocket® Premium is in place.

Art. No. 5080

60°C/140°F

**Composition:**  
77% Polyester, 23% Elastane.

**Care Handling:**  
Best durability achieved by washing at 60°C/140°F in washing bag followed by gentle tumble-drying. Use of chlorine based bleach or chlorine based disinfection will decrease durability.

**Please keep this instruction for further reference.**

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