# Corsinel®

## by Tytex

3412 Belt with Panel

**Ostomy and Hernia Support | دعم حالات الفغر الجراحي والفتاق | Podrška za stomu i kilu | Podpurné prostriedky pro stomii a kýlu**  
**Stomi- og brokstøttbælte | Stützprodukte für Stomata und Hernien | Faja para hernia y ostomía | Stoomi ja songa tugi | Avanne- ja**  
**tyrätuki | Soutien pour stomies et hernies | Sztóma- és sérvtartó | Stuðningur fyrir stóma og kvíðslit | Supporto per stomia ed ernia**  
**トミーとヘルニアサポート | 장루 및 탈장 서포트 | Stoma- en herniaondersteuning | Stomi og brokkstøtte | Podtrzymanie stomii**  
**oraz przepukliny | Suporte para Ostomizados e Hérnia | Поддерживающие средства при стоме и грыже | Opora stómie a hernie**  
**Stomi- och bräckstöd | 造口和疝气支撑**

Instructions for Use | لامعتسال تامي لعت | Uputstvo za upotrebu | Návod k použití | Brugsanvisning | Gebrauchsanweisung  
Instrucciones de uso | Kasutusjuhend | Käyttöohje | Instructions d'utilisation | Használati utasítás | Leiðbeiningar um notkun |  
Istruzioni per l'uso | 使用说明 | 사용 지침 | Gebruiksaanwijzing | Bruksanvisning | Instrukcja użytkowania | Instruções de uso |  
Инструкции по применению | Návod na použitie | Bruksanvisning | 使用说明

EN.....	4
AR.....	6
BS.....	8
CS.....	10
DA.....	12
DE.....	14
ES.....	16
ET.....	18
FI.....	20
FR.....	22
HU.....	24
IS.....	26
IT.....	28
JA.....	30
KO.....	32
NL.....	34
NO.....	36
PL.....	38
PT.....	40
RU.....	42
SK.....	44
SV.....	46
CN.....	48
<b>T</b> 1-2.....	50-51

**Intended purpose:** Support of a hernia to prevent protrusion of the abdominal content and to retain the reduced hernia within the abdominal cavity.

**Intended users:** Users with a colostomy, an ileostomy, or an urostomy.

### Indications

For post-surgical use after construction of a colostomy, an ileostomy or an urostomy.

For management of parastomal- or other abdominal hernias

For use with both 1- and 2-piece systems

 Use the device according to the indications to avoid injury or damage.

### Clinical benefits

- Provides hernia support
- Relieves dragging sensation from hernia
- Assists mobility

### Contraindications

Absolute contraindications:

- Pregnancy
- Acute abdomen
- Abdominal compartment syndrome
- Ascites
- General discomfort/pain in the abdominal region

Relative contraindications:

- Skin diseases
- Contact dermatitis
- Chronic obstructive pulmonary disease (COPD)

### Material Composition 1

ART.NO	PRODUCT	MAIN FABRIC COMPOSITION
3412	Belt with Panel	76%PA 24%EL

Raw materials and components used in the device are tested for harmful substances according to STANDARD 100 by OEKO-TEX®, product class II, and relevant Biological Evaluation (ISO 10993). The device and packaging comply with REACH.

 The device may contain softening and antistatic agents up to 1.5%

### Care instructions

 40°C or 104°F

- Close hook & loop before washing
- Make sure hooks adhere to the loop fabric only
- Wash with similar colours
- Wash in a laundry bag
- Use of chlorine based bleach or chlorine based disinfection will decrease durability

### Performance Characteristics

#### General Performance Characteristics

-  **4-way stretch:** Increased freedom of movement and supportive fit
-  **Breathable:** Air permeable and moisture absorbing
-  **Seamless:** The device is seamlessly knitted in one piece
-  **Fully openable:** Easy-to-apply pocket and wide hook-and-loop band to adjust the support level

#### Specific Performance Characteristics

- Densely knitted material for maximum support of a hernia and abdominal muscles
- Knitted in soft and comfortable material
- Silicone anti-slip pattern at the top and bottom band securing that the device stays in place
- Boat shaped back for comfortable wear and improved mobility
- Can be used with the pocket facing right or left as no difference between up and down
- Contributes to a more harmonious body contour

### Instructions for use

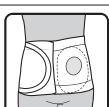
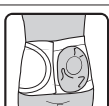
The device in primary packaging is prepared for direct clinical use.

#### How to apply the Corsinel Belt with Panel

STEPS	INSTRUCTION
1.	 <p>Apply the belt while lying or standing. Place your hand in the pocket as shown. Place the panel over the stoma site. Close the belt, making sure that the hook and loop is fastened tight and evenly.</p>
2.	 <p>Make sure the belt is not too tight or loose or has folds. Ensure that the belt fits comfortably, in both a standing and sitting position.</p>

## How to construct the individual pouch hole with Panel

These instructions are not an alternative to health professional advice.

STEPS	INSTRUCTION
1.	 <p>The following items are required:</p> <ul style="list-style-type: none"> <li>• Corsinel Belt with Panel</li> <li>• stoma pouch</li> <li>• cutting template</li> <li>• scissors</li> <li>• pen</li> </ul>
2.	 <p>Fit the belt so that the panel is located directly over the stoma pouch.</p>
3.	 <p>Find the middle of the stoma site and mark the spot on the belt with a small X. Remove the belt.</p>
4.	 <p>Remove the paper from the reverse side of the cutting template.</p>
5.	 <p>Place the cutting template loosely on top of the stoma plate, so the X on the cutting template covers the centre hole in the plate. Draw a circle on the cutting template corresponding to the circumference of the stoma plate coupling (if 2-piece) or welding between pouch and plate (if 1-piece). If 1-piece, it may be helpful to draw the circumference of the welding on the stoma plate first, before you draw the circle on the cutting template. Remove the cutting template from the stoma plate.</p>
6.	 <p>Fixate the cutting template securely on the belt so that the two X are placed on top of each other (X on the cutting template and X on the belt).</p>
7.	 <p>Make a hole with the scissors close to the centre and cut along the circle, you drew on the cutting template, but ½-1 cm smaller, as the hole will be stretched when the belt is worn. If needed you can always cut the hole bigger.</p>
8.	 <p>Remove the cutting template.</p>
9.	 <p>Apply the belt so that the hole is located directly over the pouch.</p>
10.	 <p>Gently pull the pouch through the hole.</p>
11.	 <p>Corsinel belt with individual hole for the pouch is now in place.</p>

## Measuring size

With a hernia:

- Use a standard measuring tape
- Measure yourself when lying down and when hernia has retracted, if possible
- Measure yourself at the widest abdominal circumference covered by the device
- Be sure to relax the abdominal muscles during the measurement
- Mark your measurement result on the measuring tape with two fingers (tweezers) and then remove the measuring tape to read your result. It is important that you do not lift your head while lying down to read the result, as this will give an incorrect measurement

Without a hernia:

- Measure yourself in standing position and at the widest abdominal circumference covered by the device.

Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size for an extra tight-fitting device and a larger size if you prefer the device looser.

Ensure that the device feels comfortable to wear when sitting and standing.

## Recommended size chart

- Use the size guide to find the right size based on your measurement. ① 2

## Warnings and Precautions

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to avoid injury or damage
- Make sure the device is not compromising stoma and/or lung functions
- If your hernia is irreducible (cannot be returned by manipulation), seek the advice and guidance of your ostomy nurse before using the device
- If any unexpected discomfort, stoma or skin irritation occurs, remove the device
- By urgent need for removal of the device, open with scissors

## Reporting obligation

If you have any complaints, please contact your supplier directly. In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

## Lifetime of the device in use

The lifetime of the medical device is determined by natural wear and tear, even when treated correctly and as recommended. In case of signs of deterioration affecting performance and comfort, the device should be removed.

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Corsinel is a registered trademark of Tytex A/S © 2023.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast
Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • [www.tytex.com](http://www.tytex.com)