## LEGGYFIX® Leg Bag Holder

### EN Leg Bag Holder

#### Indication:

Medical device for retention of urine leg bag.

#### Instruction for use:

1. Pull the colour-coded end of the sleeve over the leg.
2. Pull the tap through the special hole in the LeggyFix® red line.
3. Pull up the end of the sleeve over the leg bag.
4. The LeggyFix® is in place.

#### Features:

Leggyfix® is folded up around the urine leg bag to provide a pocket for the urine bag. When the sleeve is folded, a slit at the bottom will appear, and the valve of urine bags with bottom tap opening can be pulled through. The sleeve can contain urine leg bag of volumes up to 750 ml/24 oz. Suitable for use on both thigh and calf. With outer seams.

#### Precautions:

It is recommended to empty the urine bag when two-thirds full. If you have any questions regarding use and/or correct fitting, or if any unexpected discomfort or skin irritation occurs,

please consult your healthcare professional.

#### Care Handling:

Best durability achieved by washing at 60°C/140°F in washing bag followed by gentle tumble-drying. Use of chlorine based bleach or chlorine based disinfection will decrease durability. Please keep this instruction for further reference.

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